



# BASIC FUSION

#12 Rx VARIABLE VECTOGRAM \*

STEREO OPTICAL CO.

This simple group of forms is arranged to permit you to be quickly aware of the quality of binocular fusion you can achieve at various separations of the pictures.

Place the two pictures in the holder at the recommended positions and attempt to see all the forms in their proper places, not letting them disappear. Within each rectangle the square and the upper dot are seen by the right eye, the circle and the lower dot by the left, the X by both. If the square disappears, cover the left eye until you see it again and consciously try to hold it as you uncover your eye. If the circle disappears, cover your right eye momentarily until you can force a simultaneous awareness of all the forms.

The small rectangle should be used at the near reading distance, the larger one permits you to move back some distance. The two circles above have smaller circles inside which, if "fused" properly, will appear to be in front of or behind the larger circles. This is a stereoscopic clue which, if present, assures you that your mind is responding to the two images from the eyes correctly.

**NOTE:** Please store your Vectogram in a cool, dry place when not in use. High heat & humidity may cause fading.